



PLEASE NOTE THE MEDICATION CUP IS A REPLACEABLE PART
ITS LIFE IS VERY DEPENDENT ON THE DRUG NEBULIZED AND HOW WELL IT IS MAINTAINED
On average 120 uses can be expected from the Medication Cup, depending on what medication is being nebulized.



This chart is meant to be used as a guide for you and your veterinarian. Some of the medications listed are not available in all countries.
 It is recommended to always consult your veterinarian for advice regarding diagnosis and the treatment protocol for your horse.

Antibiotics / Antifungal

Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Standard Med Cup Color Code *	Special Med Cup Color Code for Shorter Delivery Time *
Cefquinome	Cobactan®	Cobactan® 4.5% Injectable Solution	No	GREY	GREEN
Ceftiofur Sodium	Excenel®	Excenel® 4G Injectable Solution	No	GREY	GREEN
Gentamicin	Gentaject	Gentaject 10% Injectable Solution	Yes 1:1	GREY	GREEN
Amikacin	Amikavet	Amikavet Injectable Solution	Yes 1:1	GREY	GREEN
Marbofloxacin	Marbocyl	Injectable Solution	Yes 1:1	GREY	GREEN
Nystatin	R-Nystatine	Oral Solution	Yes 1:1	GREY	GREEN

Bronchodilators

Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Standard Med Cup Color Code *
Ipratropium	Atrovent®	Atrovent® 250 UDVS Nebulizer solution	Yes 1:1	GREY
Salbutamol (Albuterol)	Ventolin™	Ventolin™ 2.5 Nebulizer Solution	Yes 1:1	GREY
Clenbuterol	Ventipulmin®	Ventipulmin® 30 Injectable Solution	Yes 1:1	GREY

Corticosteroids

Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Standard Med Cup Color Code *
Dexamethasone	Dexameth	Dexameth 2mg/ml Injectable Solution	Yes 1:1	GREY
Fluticasone Propionate	Flixotide™	Flixotide™ 2mg/2ml Nebulizer Suspension	Yes 1:1	GREY
Budesonide	Pulmicort®	Pulmicort® 0.5 Nebulizer Solution	Yes 1:1	GREY

Mucolytics


Drug	Trade Name	Drug Format	Dilute with 0.9% Saline	Standard Med Cup Color Code *	Special Med Cup Color Code for Shorter Delivery Time *
Acetylcysteine	Parvolex®	Parvolex® 2g Solution for infusion	Yes 1:1	GREY	GREEN
0.9% Saline Solution	0.9% Saline Solution	B.Braun NaCl 0.9%	No	GREY	None
Bromhexine	Bisolvon	Oral Solution	Yes 1:1	GREY	None
Emser Salt	Emser Salt	Nebulizer Salt Solution	No	GREY	None

Other Therapies

Active Ingredients	Trade Name	Format	Dilute with 0.9% Saline	Standard Med Cup Color Code *	Special Med Cup Color Code for Shorter Delivery Time *
Liquid silver and distilled water	Various Brands	Oral Solution	No	GREY	None
Thymus proteins, zinc gluconate, silver solution	BioEqualizer	Oral Solution	Yes 1:1	GREY	None
Hyaluronic acid, eucalyptus essential oil in isotonic solution	Hyalusol®	Nebulizer Solution	No	GREY	None
Natural solutions including honey and bryonia cretica	Larynx/Apis	Oral Solution	Yes 1:1	GREY	None
Mixture of essential oils	Inhal-14	Nebulizer Solution	Yes 1:1	GREY	GREEN
Grindelia, water and essential oils	Balsamic Air	Oral Solution	Yes 1:1	GREY	GREEN
Urtica dioica, vitis vinefera, avena sativa	Epistop-L	Nebulizer Solution	Yes 1:1	GREEN	None
Mixture of essential oils	StopBlood	Oral Solution	Yes 1:1	GREEN	None

* Read the reverse side of this Drug Table for important information and advice on Med Cup & Drug combination choices

Med Cup Color Code Guide

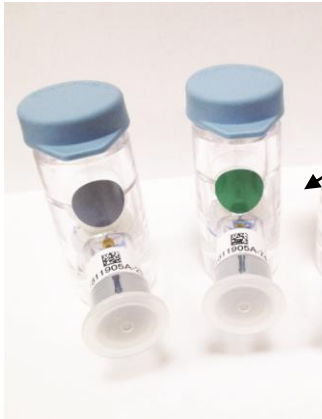

- The medication cup is coded with a Serial Number & Date of Manufacturing on the connector
- The label can be colored Grey or Green
- The Grey labeled med cup is the standard cup that will nebulize most medications
- A Green labeled med cup is a faster cup for shorter delivery time with some medications & natural therapies

Important Information:

The med cup & drug combination choice can have a big impact on the delivery time and aerosol droplet size.

- A **GREY** labeled cup used with high viscosity oily type drugs can lead to slow delivery or failure to nebulize.
- A **GREEN** labeled cup used with low viscosity watery type drugs can lead to very fast delivery and the generation of big aerosol droplets leading to pools of medication wasted in the bottom of the mask chamber.
- If the drug you wish to nebulize is not listed, please contact Nortev or your local distributor prior to nebulizing for advice as drugs not listed for use may cause damage to your medication cup
- If you are unsure about what med cup to use it is best to contact your local distributor for advice.
- Check out our website at www.nortev.com for any updates to this information.