



YOUR HORSE

Why use Flexineb in the Winter?

Your horse spends more time indoors.

Your horse is indoors more, hence will be **prone to dust and other airborne particles**, which is directly linked to the increase in Respiratory issues.

While indoors, your horse is in close proximity to other equines, and hence **susceptible to common respiratory ailments** such as, Bacterial, Viral and Fungal Infections.

Higher risk of respiratory health issues.

Studies have shown that Mild to Moderate Equine Asthma is nearly **30% more prevalent in the winter months** compared to the Summer.

Stabling associated with winter can worsen already existing or trigger off respiratory issues such as the above mentioned **Equine Asthma and Respiratory Infections** (Bacterial, Viral and Fungal).

Research has shown that **cold weather exercise** can cause Equine Asthma in performance horses. Repeated work in cold temperatures can lead to chronic airway inflammation.



Flexineb can be used to help treat and maintain common equine respiratory issues during the winter months with a wide variety of Drugs and non-doping natural products, such as SilvaPlex and Immuno San.

 **Flexineb**[®]

Phone: 353 (0) 91 799778
Email: info@nortev.com

Flexineb.com



YOUR FLEXINEB

How to use Flexineb in the Winter?

Adjust Temperature

To expedite nebulization, bring the solutions to room temperature (18-20°C, 64-68°F).

Battery Use & Charge

Charge your battery indoors for at least 6 hours.

Proper Storage

Store your Flexineb in a clean, warm and dry environment.

Natural Solutions

Utilize solutions like SilvaPlex with short or no withdrawal times.

Consistent Cleaning

Properly clean and disinfect the mask to reduce cross contamination.

Med Cup Maintenance

Properly dry the med cup after each treatment and clean the cup daily to prevent clogging and freezing.



Flexineb can be used to help treat and maintain common equine respiratory issues during the winter months with a wide variety of Drugs and non-doping natural products, such as SilvaPlex and Immuno San.

 **Flexineb**[®]

Phone: 353 (0) 91 799778
Email: info@nortev.com

Flexineb.com

